


TREAT HER RIGHT. TREAT HER ONCE.

Accu-bloc[®]

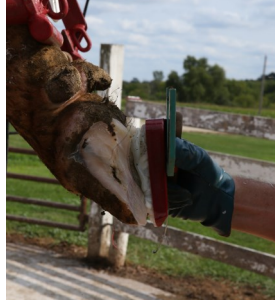

Accu-sole[™]



APPLICATION STEPS

STEP 1: Trim healthy claw and follow by prepping with rasp to clean and texture surface.

STEP 2: Dry and warm healthy claw and block with a heat gun or propane torch.

STEP 3: Heat block to enhance integrity of the bonding surface.

STEP 4: Apply 30-50cc of *Accu-bond* onto warm block.

STEP 5: Position block on prepared claw. Ensure 1/8" - 3/16" (3-5 mm) of *Accu-bond* between block and bottom of claw. Hold until bond begins to cure and then press firmly for 10 seconds. Allow *Accu-bond* to cure for three minutes before releasing leg and making contact with ground.