# CLAW BLOCKING TECHNOLOGY





### STORAGE

Keep *Accu-bond* around 70°F/21°C, out of direct sun. Store no longer than 12 months.

#### **GLUE TEMPERATURE**

Cold Weather = Raise **Accu-bond** temperature to  $65^{\circ}F/18^{\circ}C$ . Hot Weather = Lower **Accu-bond** temperature to  $65^{\circ}F/18^{\circ}C$ . Changing the temperature will give user adequate time for application and superior bond.

#### **GLUE TIPS**

- 1. Prepare both block and claw surface using a rasp and
- 2. 200mL cartridges interlock with the dispenser/applicator at the neck of the cartridge.
- 3. Prime new cartridge to remove air and equalize both sides.
- 4. Keep cartridge upright after tip change.
- 5. Begin dispensing in upright position until mixing tip is full.
- 6. Release dispenser pressure after each use.
- 7. Leave tip in place until next application
- 8. Clean all dried glue from neck of cartridge before attaching next tip.

## **APPLICATION STEPS**

STEP 1: Trim healthy claw and follow by prepping with rasp to clean and texture surface.

STEP 2: Dry and warm healthy claw and block with a heat gun or propane torch.

STEP 3: Heat block to enhance integrity of the bonding surface.

STEP 4: Apply 30-50cc of Accu-bond onto warm block.

*STEP 5:* Position block on prepared claw. Ensure 1/8"- 3/16" (3-5 mm) of *Accu-bond* between block and bottom of claw. Hold until bond begins to cure and then press firmly for 10 seconds. Allow *Accu-bond* to cure for three minutes before releasing leg and making contact with ground.







